MFJ-388 Sound Magnifier Instruction Manual

WARNING: The MFJ-388 Sound Magnifier is not a hearing aid for those with impaired hearing. If you have impaired hearing consult your doctor.

It is important to increase the volume control very gradually to avoid sudden increase in sound.

ON / OFF SWITCH

The correct positions of the switch are as shown below:



The switch should be turned to the "OFF" position when the MFJ-388 is not in use. This will ensure longer battery life.

VOLUME CONTROL

There are 6 levels of volume control. It is recommended that the volume level always be on Volume Level 1 when the unit is turned on. The volume should then be adjusted to a suitable level.

BATTERY INSTALLATION

Open the battery hatch. Insert the battery with the + side up.

BATTERY OPTIONS

The MFJ-388 Sound Magnifier uses a 1.5 V mini button-cell battery. These are available at many electronic stores, drug stores, or anywhere watch batteries are sold. The following is a list of batteries that can be used in the MFJ-388 Sound Magnifier:

EVEREADY 357	AWI SO5
BULOVA 228	DURACELL D357
MAXELL SR44W	PANASONIC SP35
R-O-V 357	RENATA 7
SEIKO SB-89	TIMEX J
VARTA B357	RADIO SHACK 76
8006, EPX 76	S76, GP76
GP, WL14	G13

CAUTION

This product is not waterproof. Therefore, the user must take care to prevent water from entering the MFJ-388 Sound Magnifier. If water does enter the MFJ-388. remove the battery and let the water dry before further use.

INSTRUCTIONS FOR USE

- Turn the volume to level 1. 1.
- 2. Check the battery to see that it is properly inserted.
- 3. Place the MFJ-388 Sound Magnifier into your ear, making sure that the earphone

is inserted deeply into ear canal before switching the unit on.

- 4. Use the MFJ-388 Sound Magnifier only when situations which require amplification of sounds, such as movies and television. The MFJ-388 Sound Magnifier is not intended to be worn for more than several hours at a time. Excessive use will drain the batteries. Turn the MFJ-388 Sound Magnifier off when not in use.
- Turn the switch to the "ON" 5. position to activate the MFJ-388 Sound Magnifier. You should hear an air flow noise.
- Gradually increase the volume 6. until it is at a suitable level. Volume level 3 is suitable for most users.

rotion to denv warranty without dated proof-of-purchase. Any evidence of alti

- discretion to deny warrany without dated proof of purchase. Any evidence of alteration, ensarenc, or forgers whall be cause to vold any and all warrany terms immediately. MFJ Enterprises, Enc. agrees to repair or replace at MFJ's option without charge to the original owner any defective product under warrany, provide the product is returned postage prepaid to MFJ Enterprises, Enc. with a personal neck, cashiers check, com money order for 57.00 covering postage and handling. MFJ Enterprises, Enc. will supply replacement parts free of charge for any MFJ product under warrany upon regulates. A dual of proof defurchase and a 55.00 personal check, cashiers check, or money order must be provided to cover postage and handling. The for an ency order must be provided to its cover postage and handling. This warranty does not apply to kins sold by or manufactured by MFJ Enterprises, Inc. Wired and itsedPC logand moders: a covered by this strany trovided out due wired 2.
- 4 5. 6

7

- This warrany does not apply to kits sold by or manufactured by MP Enterprises, Inc. Wired and tested FC board products are covered by this warrany provided only the wired and tested FC board products is returned. Wired and tested FC boards installed in the owner's calinet connected to switching, back, or calibles, test to MP Enterprises, the control of the second sec
- 8.
- products previously manufactured. All MFJ products to be serviced inwarranty or out-of-warranty should be addressed to MFJ 11.
- Enterprises, Inc., 300 Industrial Park Road, Starkville, Mississippi 39759, USA and must be accompanied by a letter describing the problem in detail along with a copy of your dated proof of purchase. This warranty gives you specific rights, and you may also have other rights which vary from
- 12.

12 MONTH LIMITED WARRANTY

MFJ Enterprises, Inc. Warrants to the original owner of this product, if manufactured by MFJ Enterprises, Inc. and purchased from an authorized dealer or directly from MFJ Enterprises, Inc. to be free from defects in material and workmanship for a period of 12 months from date of purchase provided the following terms of this warranty are satisfied.

1

The purchaser must retain the dated proof-of-purchase (bill of sale, canceled check, credit card or money order receipt, etc.) describing the product to establish the validity of he warranty claim and submit the original or machine reproduction of such proofef purchase to MFJ Enterprises, Inc. at the time of warranty service. MFJ Enterprises, Inc. shall have the