

## MFJ-389 Sound Magnifier Instruction Manual

**WARNING:** The MFJ-389 Sound Magnifier is not a hearing aid for those with impaired hearing. If you have impaired hearing consult your doctor.


It is important to increase the volume control very gradually to avoid sudden increase in sound.

### CAUTION

This product is not waterproof. Therefore, the user must take care to prevent water from entering the MFJ-389 Sound Magnifier. If water does enter the MFJ-389, remove the battery and let the water dry before further use.

### ON / OFF SWITCH


The correct positions of the switch are as shown below:

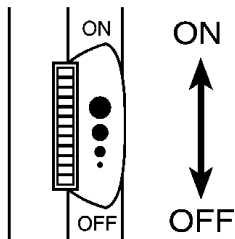

The switch should be turned to the "OFF" position when the MFJ-389 is not in use. This will ensure longer battery life.


### VOLUME CONTROL

It is recommended that the volume level always be started at the minimum level when the unit is turned on. After turning ON the MFJ-389, rotate the volume control upward to increase the volume to a suitable level. Rotate the volume control downward to decrease the volume level.

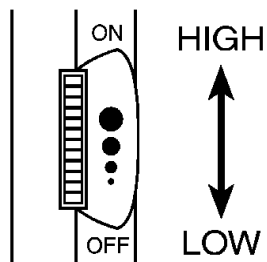


### BATTERY INSTALLATION

The MFJ-389 uses one AAA size battery (not included). To install the battery, open the battery cover. Insert the battery according to the polarity marks inside the battery compartment.

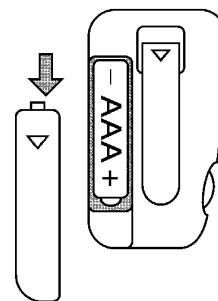

### INSTALLING THE EARPHONE

Insert the earphone plug into the jack on the MFJ-389 as shown below:

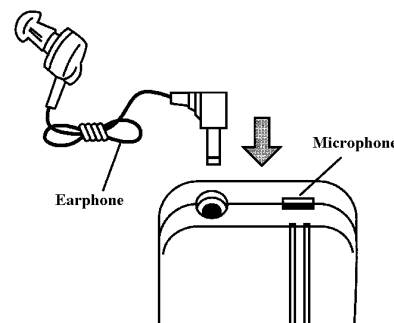

### INSTRUCTIONS FOR USE

1. Check the battery and earphone to see that they are properly inserted.
2. Place the earphone of the MFJ-389 Sound Magnifier into your ear, making sure that the earphone is inserted deeply into ear canal before switching the unit on.

3. Turn the switch to the "ON" position to activate the MFJ-389 Sound Magnifier. You should hear an air flow noise.
4. Gradually increase the volume until it is at a suitable level.
5. Once the volume level is set, place the MFJ-389 Sound Magnifier on your belt or in your shirt pocket.
6. Use the MFJ-389 Sound Magnifier only when situations which require amplification of sounds, such as movies and television. The MFJ-389 Sound Magnifier is not intended to be worn for more than several hours at a time.
7. Excessive use will drain the batteries. Turn the MFJ-389 Sound Magnifier OFF when not in use.

# 12 MONTH LIMITED WARRANTY

MFJ Enterprises, Inc. Warrants to the original owner of this product, if manufactured by MFJ Enterprises, Inc. and purchased from an authorized dealer or directly from MFJ Enterprises, Inc. to be free from defects in material and workmanship for a period of 12 months from date of purchase provided the following terms of this warranty are satisfied.

1. The purchaser must retain the dated proof-of-purchase (bill of sale, canceled check, credit card or money order receipt, etc.) describing the product to establish the validity of the warranty claim and submit the original or machine reproduction of such proof-of-purchase to MFJ Enterprises, Inc. at the time of warranty service. MFJ Enterprises, Inc. shall have the discretion to deny warranty without dated proof-of-purchase. Any evidence of alteration, erasure, or forgery shall be cause to void any and all warranty terms immediately.
2. MFJ Enterprises, Inc. agrees to repair or replace at MFJ's option without charge to the original owner any defective product under warranty, provided the product is returned postage prepaid to MFJ Enterprises, Inc. with a personal check, cashiers check, or money order for \$7.00 covering postage and handling.
3. MFJ Enterprises, Inc. will supply replacement parts free of charge for any MFJ product under warranty upon request. A dated proof-of-purchase and a \$5.00 personal check, cashiers check, or money order must be provided to cover postage and handling.
4. This warranty is NOT void for owners who attempt to repair defective units. Technical consultation is available by calling (601) 323-5869.
5. This warranty does not apply to kits sold by or manufactured by MFJ Enterprises, Inc.
6. Wired and tested PC board products are covered by this warranty provided only the wired and tested PC board product is returned. Wired and tested PC boards installed in the owner's cabinet or connected to switches, jacks, or cables, etc. sent to MFJ Enterprises, Inc. will be returned at the owner's expense unrepared.
7. Under no circumstances is MFJ Enterprises, Inc. liable for consequential damages to person or property by the use of any MFJ products.
8. Out-of-warranty Service: MFJ Enterprises, Inc. will repair any out-of-warranty product provided the unit is shipped prepaid. All repaired units will be shipped COD to the owner. Repair charges will be added to the COD fee unless other arrangements are made.
9. This warranty is given in lieu of any other warranty expressed or implied.
10. MFJ Enterprises, Inc. reserves the right to make changes or improvements in design or manufacture without incurring any obligation to install such changes upon any of the products previously manufactured.

11. All MFJ products to be serviced in-warranty or out-of-warranty should be addressed to MFJ Enterprises, Inc., 300 Industrial Park Road, Starkville, Mississippi 39759, USA and must be accompanied by a letter describing the problem in detail along with a copy of your dated proof-of-purchase.
12. This warranty gives you specific rights, and you may also have other rights which vary from state to state.